
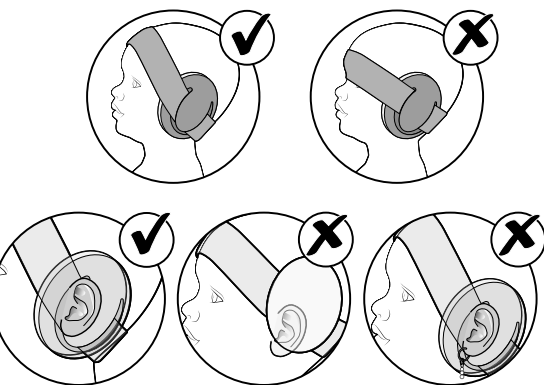
SilentGuard Baby

Art.Nr.: 53063 / 53074

Anwendung • application • solitud
applicazione • application • toepassing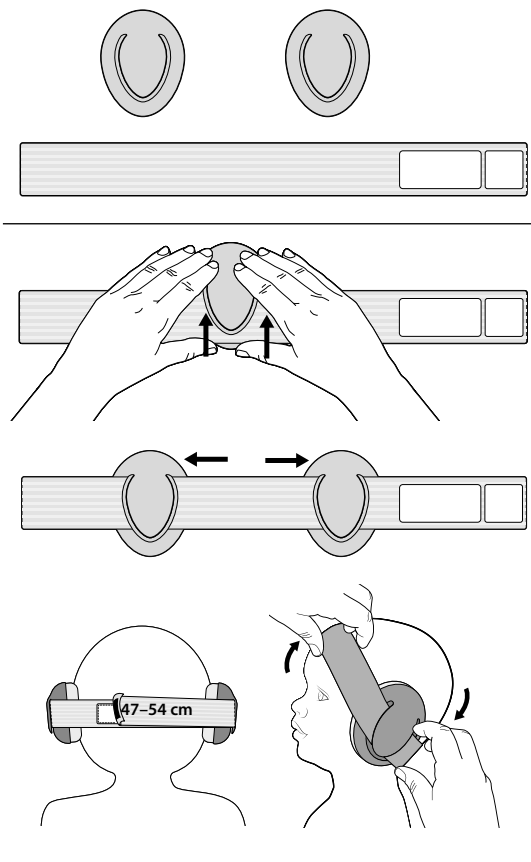

Gebruiksaanleitung

DE

Kapselgehörschutz // Art.Nr.: 53063 / 53074

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GB

Capsule ear protectors // item no.: 53063 / 53074

The SilentGuard Baby from reer protects a baby's hearing from damaging noise. Read these user instructions carefully before use. The ear protectors must be set up, adjusted and cleaned according to the instructions in this user manual. **Warning:** Failure to follow the instructions in the user manual will seriously impair the protective effect of the earmuffs.

General safety instructions:

- Check the unit for visible damage before each use. If you detect any damage, do not use the unit.
- This product is not a toy! Please use the product only for its intended purpose, and under adult supervision. Make sure that the earmuff is worn correctly and continuously by the child. The time a child stays in a noise area should be minimised.
- Keep the ear protectors dry and do not store at temperatures higher than +55 °C.

Instructions for safe use:

- Full protective effect may only be achieved if the ear protectors are used throughout the child's presence in a loud environment. Loss of hearing can occur with sounds as low as 85 dB. The maximum usage duration without a break should not exceed 90 minutes and in total, the hearing protection should not be worn for longer than 3 hours a day.
- Earmuffs and especially the sealing pads can wear out with use and should therefore be checked frequently for signs of, for example, cracks and leaks. Encasing the ear seals with „a hygienic covering“ can effect the acoustic performance of the capsule ear protection. The durability of the pillows is about 3-5 years.

Material: plastic, 100 % nylon

Average mass: 121 g

Cleaning: Regularly clean the outside of the product with soap and warm running water. The headband can be washed manually. Close the zip fastener first. This product may suffer from certain chemical substances. Further information can be requested directly from the manufacturer.

Attention! Do not submerge the ear protectors (ear capsules) in water.

Use: (see diagrams)

Fasten the ear capsules on the headband, place the headband onto the baby's head. Close the zip fastener and adjust as necessary.

Important! Optimum sound attenuation can only be achieved if the sealing rings are fitted snugly against the head, enclosing the entire ear. Pull back your child's hair to ensure this. Temples of eyeglasses must be as thin as possible, and removed whenever possible. Do not wear jewellery, e.g. earrings.

Attention! Avoid applying too much pressure on the parts of the head that are still soft (fontanel), because the baby's head is still developing.

Attenuation value: (see table)

The reer SilentGuard baby is according EN 352-1:2002 tested and monitored by PZT GmbH, Bismarckstrasse 264 B, 26389 Wilhelmshaven, Germany. Notes on the attenuation table: 1) The frequency is in Hz. 2) The attenuation average is in dB. 3) The standard deviation is in dB. 4) Assumed protection value 5) Calculated attenuation value

Please retain your receipt, the user manual and item number. Warranty claims or claims from promises of guarantee must be processed by the dealer. reer GmbH hereby declares that the PSA SilentGuard Baby (item no.: 53063 / 53074) complies with the regulation (EU)2016/425. The full text of the EU Declaration of Conformity can be found at the following website: <http://www.reer.de>

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